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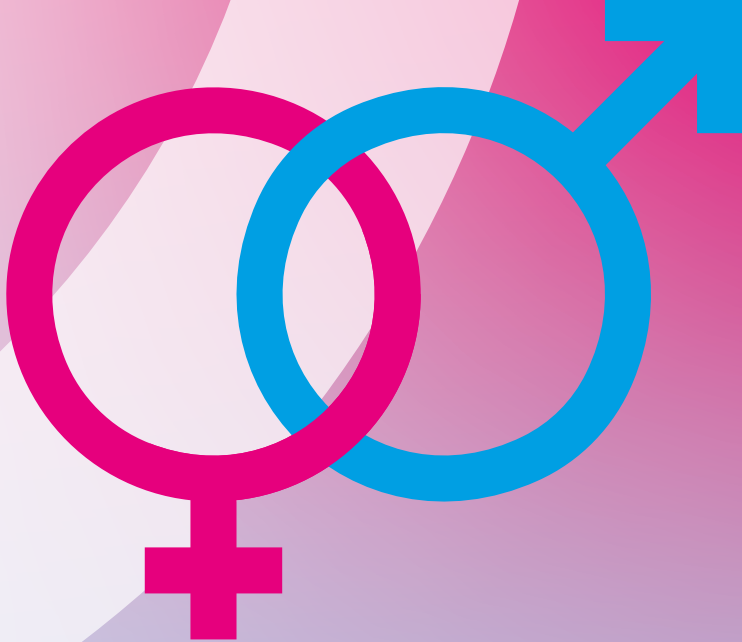
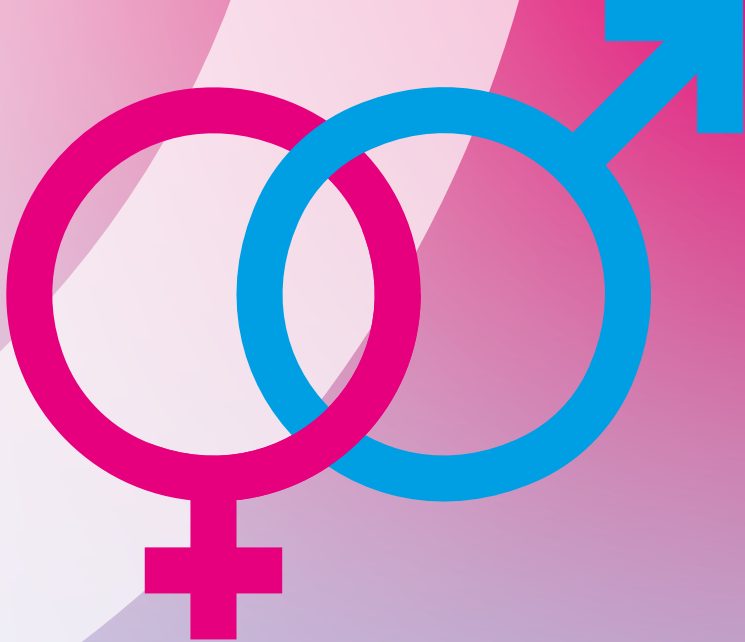
ABOUT THIS TEST

This kit is for you to take a blood sample, which is sent to the certified Microbe&Lab Laboratory for analysis.

The test determines whether the tested Sexually Transmitted Diseases (STD) are present in your sample. STDs are often accompanied by complaints such as pain and a burning sensation when urinating, discharge from the urethra (especially in the morning), pain in the scrotum, inflammation of the urethra, foreskin and/or prostate and/or other inflammations in the body.

This test is suitable for men and women who want to know if they test positive for STDs.

If your test result shows that you are STD positive, it is important that you and your (former) sexual partner or partners are tested and, in case of a positive test, treated. Untreated, STDs can lead to, among other things, infertility, ectopic pregnancies and/or infections of the uterus, this goes for you and/or your (former) sexual partner or partners.

In case of a positive result, we are happy to assist you with the treatment by engaging our BIG-registered doctors who collaborate with Apotheek De Bosrand.

The following applies to this test: Consult a doctor before using this test if you suffer from haemophilia or if you are taking anticoagulant medication • You may experience a dizzy feeling when handling blood • Never use a lancet previously used by another person • Do not use this test internally • Do not touch the blue button before you are ready to prick • Keep the used test away from others • If the lancet is not disposed of properly, it may cause injury or infection • Keep out of the reach of children • Store dry and at room temperature • All components are for single use • Do not use after the expiry date • Do not use with damaged contents.

Check carefully that all components shown below are present before taking the test. Despite extensive quality control, it may happen that a component is missing. In that case, please contact us via hulp@ikwilvanmijnsoaaf.nl

INSTRUCTIONS FOR USE

Step-by-step plan to collect and send sample

STEP 1
Have you not eaten or drunk anything for **at least 10 hours**? Then you are now ready for the test! Check the contents of the package. Place the **collection tube A**, the **lancet B**, and the **patch D** ready to use. The test kit contains a **code card E** with a unique code. **You need this code to request your result.** Keep the code card safe and take a photo to be sure.

STEP 2
Before using the test, wash your hands with soap and warm water. Dry your hands thoroughly. **Warm the fingers and hands for 30 seconds**, either by massaging them or by warming them under lukewarm water ($\pm 39^{\circ}\text{C}$).

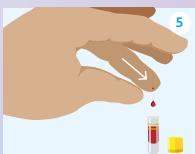
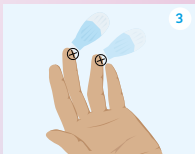
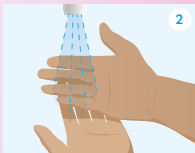
Important: the finger should be warm before pricking blood. Make sure you get a good test, take your time for this.

STEP 3
Decide on which finger and where on the finger (middle or ring finger) you want to prick. The best place is on the side of the finger. Give the finger another good massage.

STEP 4
Twist the cap of the **collection tube** and set the collection tube upright or use the holder in the return box. Now twist the cap of the **blood collection lancet** and place it on the side of the tip of your finger. Press the lancet onto your finger with a smooth motion. The needle automatically retracts.

STEP 5
Massage the pricked finger from the palm of your hand towards your fingertip, this will keep enough drops forming, thus collecting at least 0.5 ml of blood (about 15–20 drops) into the blood collection tube so that the blood **reaches at least the bottom line** on the tube.

Not enough blood? If necessary, repeat 'Step 5' until you have collected enough blood.
If there is not enough blood in the tube, no analysis can be conducted.

Below are the instructions for use to take a blood sample. It is important to take a moment for this test and to read the instructions carefully. It will take about 20 minutes.

In addition, a few important tips:

1. Make sure that you fill the blood collection tube at least to the bottom yellow line on the tube. Is there less blood in it? Then our laboratory is probably unable to conduct a proper analysis.
2. Make sure to seal the blood collection tube tightly after the blood collection.
3. Keep the code card from the test kit - with your unique code - safe. Because only with this unique code can you request your personal result. (Tip: take a photo of the code card!)
4. Some tests need to be taken on an empty stomach. You should therefore read the step-by-step plan carefully before starting to take the test.

! Read this leaflet completely prior to starting your test.

STEP 6
Make sure you seal the collection tube properly with a **click sound** after the blood collection.

STEP 7
Stick the plaster on the pricked spot.

STEP 8
Take the well-sealed collection tube and **tilt it 10 times**. Do not shake it. The transparent liquid in the tube does not need to mix with the blood.

STAP 9
Place the blood collection tube with the **absorbent material C** in the **safety bag F**. Remove the white foil strip from the adhesive edge and close the safety bag tightly.

STEP 10
Place the safety bag in the **postage-paid return box G**.
Write your **collection date** on the sticker provided on the code card and stick it to the inside of your postage-paid return box.


STEP 11
Close the postage-free return box tightly and then seal it with the **seal sticker** provided on the code card. The postage-free return box already has a return address with a reply number. **No need** for stamps.

Your sample is now ready for free postage via a PostNL letterbox or post office. Postage preferably on Sunday to Friday (before the collection at 5:00 PM).

Attention! Keep the **code card E** with the unique code safe. The code is necessary to retrieve your results online.

INCUBATION PERIOD

Chlamydia, Gonorrroe, Mycoplasma	7-14 days
Trichomonas.....	21 days
Syphilis, HIV.....	4 weeks
Hepatitis B/C.....	8 weeks
HPV	several months/years

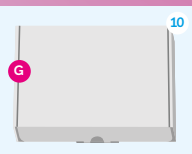REQUEST YOUR RESULTS

As soon as your sample arrives at our laboratory, it will be processed the same day. On weekdays, you can view your results online from 6:00 PM.

Go to ikwilvanmijnsoaaf.nl and click on "Mijn uitslag". Enter the unique code on your code card and download your results report.

Do you see the message "Sorry, er zijn geen resultaten gevonden"? This means that your sample has not yet arrived at the laboratory. Please wait a bit, the result will be available as soon as your sample is processed.

Do you have any more questions?
We will gladly assist. Please contact: hulp@ikwilvanmijnsoaaf.nl

Important!
Keep your unique code safe!
Take a photo of it if necessary.
No results without code!

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