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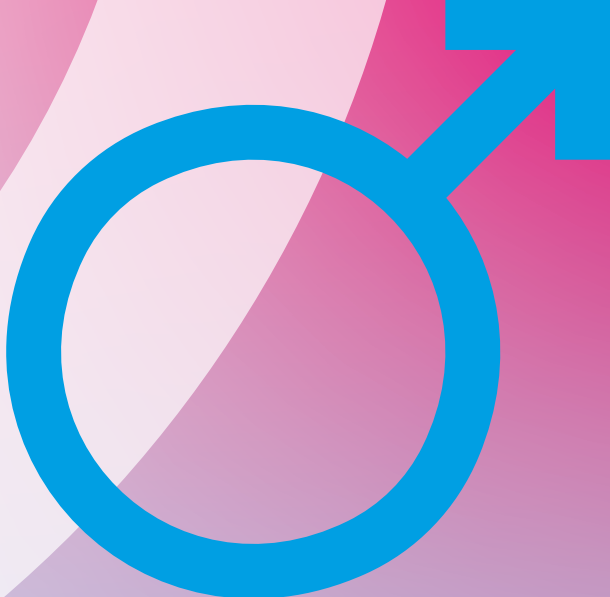
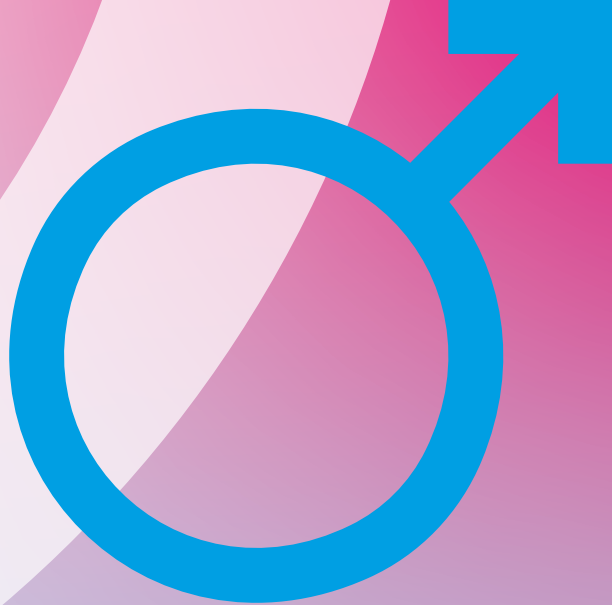
ABOUT THIS TEST

This kit is for you to take a urine sample, which is sent to the certified Microbe&Lab Laboratory for analysis.

The test determines whether the tested Sexually Transmitted Diseases (STD) are present in your sample. STDs are often accompanied by complaints such as pain and a burning sensation when urinating, discharge from the urethra (especially in the morning), pain in the scrotum, inflammation of the urethra, foreskin and/or prostate and/or other inflammations in the body.

This test is suitable for men who want to know if they test positive for STDs.

If your test result shows that you are STD positive, it is important that you and your (former) sexual partner or partners are tested and, in case of a positive test, treated. Untreated, STDs can lead to, among other things, infertility, ectopic pregnancies and/or infections of the uterus, this goes for you and/or your (former) sexual partner or partners.

In case of a positive result, we are happy to assist you with the treatment by engaging our BIG-registered doctors who collaborate with Apotheek De Bosrand.

! Read this leaflet completely prior to starting your test.

CONTENT OF PACKAGING

! **PLEASE NOTE:** carefully check if all items specified below are present before you take the test. Despite extensive quality control, it is possible that a part is missing. In that case, contact us via hulp@ikwilvanmijnsoaf.nl

A Urine pipette
Use the pipette to collect urine and place it in the urine collection tube.

B Urine collection tube
Place your urine into the tube via the urine pipette and close it tightly.

C Code card
Keep the **code card** with your unique code in a safe place! This is necessary to request your results online. **No results without code!**

D Absorption material
Place the urine collection tube in this.

E Safety bag
Place the sample material in the bag, then in the return box.

F Postage-paid return box
Label the box with the collection date and close it with the closing sticker, both present on the code card.

The following is applicable to this test: keep out of reach of children. Store it dry and at room temperature. All parts are for once-off use. Do not use it after the expiry date. Do not use if the content has been damaged.

INSTRUCTIONS FOR USE

Step-by-step plan to collect and send sample

STEP 1
Urinate in a glass, preferably in the morning after getting up (at least 2 hours without urinating). It is very important that you collect a small amount (15-20 ml) of the first stream of urine in the glass.

STEP 2
Take the enclosed **urine pipette** **A** and use it to suck approximately 1.25 ml of the urine out of the glass. Use the **urine pipette** to transfer the aspirated urine into the enclosed **urine collection tube** **B** and close it tightly.

STEP 3
Place the urine collection tube in the **absorption material** **D** and put it in the **safety bag** **E**. Remove the white foil strip from the adhesive edge and seal the bag tightly.

STEP 4
Place the **safety bag** in the **postage-paid return box** **F**. Write the sample date on the **code card** sticker and stick it on the inside of the box.

STEP 5
Close **the box** properly and use the sealing sticker from the **code card** to secure the box. The return address with a freepost number is already prepared for you, so no need to add any stamps. You can drop your sample at the nearest PostNL letterbox or post office.

INCUBATION PERIOD	
Chlamydia, Gonorrhoe, Mycoplasma	7-14 days
Trichomonas.....	21 days
Syphilis, HIV.....	4 weeks
Hepatitis B/C.....	8 weeks
HPV	several months/years

Attention! Keep the **code card** **C** with the unique code safe. The code is necessary to retrieve your results online.

REQUEST YOUR RESULTS

As soon as your sample arrives at our laboratory, it will be processed the same day. On weekdays, you can view your results online from 6:00 PM.

Go to ikwilvanmijnsoaf.nl and click on "Mijn uitslag". Enter the unique code on your code card and download your results report.

Do you see the message "Sorry, er zijn geen resultaten gevonden"? This means that your sample has not yet arrived at the laboratory. Please wait a bit, the result will be available as soon as your sample is processed.

Do you have any more questions?
We will gladly assist. Please contact: hulp@ikwilvanmijnsoaf.nl

! **Important!**
Keep your unique code safe!
Take a photo of it if necessary.
No results without code!